Zadelhoogte

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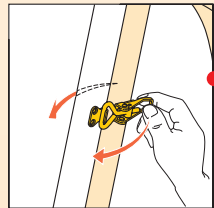
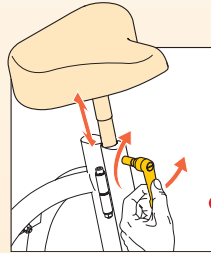
Height of the saddle

- Open the quick clamp a few turns
- Adjust the saddle to the correct height
- Close the quick clamp

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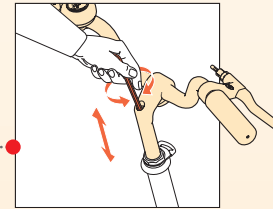
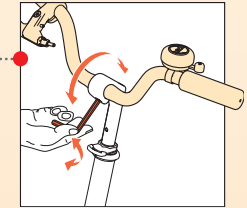
Height of the handle bar

- Loosen the bolt with an Allen key a few turns
- Adjust the handle bar to the correct height
- Tighten the bolt with an Allen key

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Inclination of the handle bar

- Loosen the bolt with an Allen key a few turns
- Adjust the handle bar to the correct position
- Tighten the bolt with an Allen key

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MAINTENANCE SCHEDULE (TO BE EXECUTED BY A DEALER)

Interval	Checkpoints
After 200 km, or after 2 months	The brake
Every 12 months	The whole bike
After a fall, a collision or an accident	The whole bike

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Van Raam declares as manufacturer that the walking support bicycle has been produced in accordance with the 93/42/EEG class 1 guidelines. All products have been produced in accordance with the 2006/42/EG guidelines. The declaration of conformity can be found on the Van Raam website.

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TECHNICAL DETAILS

Length	113 cm
Width	55 cm
Height	96 cm
Front wheel	12,5"
Rear wheel	12,5"
Weight (kg)	9 kg
Maximal user weight	100 kg
Brakes front	Cantilever brakes

MANUFACTURER OF ADAPTED BICYCLES AND FRAMES

Van Raam B.V.
Aaltenseweg 56
7051 CM Varsseveld
The Netherlands
T +31 (0)315 25 73 70
E info@vanraam.com
I www.vanraam.com

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ATTENTION

Check every time before you go for a ride:

- the brakes;
- the tire pressure;
- that all parts are fitted properly.

Do not use a pressure washer or steam cleaner to clean the bike.

The bike is designed for normal use on level and solid ground.

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The guarantee is valid for all faults in the product, which are proven to be caused by material or manufacturing error.

We give a guarantee of five years on the frame and the front fork. For all accessories and other parts (except wear parts) we provide a two year guarantee, as legally regulated.

The guarantee as well as the product liability in general expires if the instructions stated in this manual are not followed. This is also the case if unauthorized maintenance is carried out or if technical changes or additions are made without original Van Raam parts or without the permission of Van Raam. Also the original CE-marking of the bike will expire.

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